

**skin condition**

Normal to mature or prematurely-aging skin.

**description**

A unique tool that offers microneedling and nanoinfusion options. Multiple cartridges offer customization options. Pair it with a professional serum to deliver visible results.



professional only



# pro pen

**application****microneedling and nanoinfusion**

1. Choose cartridge, professional serum or exfoliant, needle or nano tip, depth and stitch frequency depending on targeted concern.
2. Apply skin-specific professional serum or exfoliant.
3. Perform the microneedling or nanoinfusion technique.
4. Remove serum or exfoliant with a damp esthetic wipe and remove exfoliants with a double cleanse.
5. Prescribe home care to achieve best results.

**4 cartridge options allow for customization for every client****• 9-Pin**

9 stainless-steel needles on square plate (5x5mm). This provides the most advanced type of treatment the device offers for exfoliation, as well as smoother and firmer skin.

**• 24-Pin**

24 stainless-steel needles tightly stacked together on a round plate (7.93mm diameter). This delivers a slightly stronger exfoliation and smoother skin.

**• Nano Metal**

Fine cones on a stainless-steel plate (5x5mm metal). The larger plate helps cover larger areas. It is designed for a slightly stronger exfoliation and helps with skin luminosity.

**• Nano Silicon**

Fine cones on small silicon plate (2.87x2.87mm). Very gentle, it is designed to delicately exfoliate when skin is somewhat sensitive.